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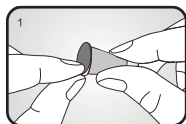
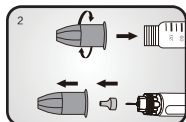
EN
Instructions
English

SV
Instruktioner
Swedish

ZHT
使用說明書
Traditional Chinese

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1. Peel off the sealing paper.

2. Holding the outer cap, vertically screw the pen needle onto the pen, and then pull the outer cap straight off. Then remove the inner cap.

3. Always check the flow of the pen needle before each injection. Refer to the procedure described in the instruction for use of the delivery device. A drop of liquid should appear at the needle tip. If not, repeat until liquid appears at the needle tip. **Note:** Liquid may fall back into the pen needle which may result in fluid being left on the skin after injection.

4. Set the dose prescribed by your doctor following the instruction for use of the pen injector. Please consult your doctor which injection method is most suitable for you.

4A – NO PINCH METHOD
Hold the pen with one hand, in palm. Your thumb should not be on the button of the delivery device. Insert the pen needle STRAIGHT into the skin.

4B – PINCH METHOD
To avoid injecting into the muscle (e.g., young children, low body fat), or to ensure a firm skin surface for a successful injection, lift a 1 inch (2.5 cm) skinfold (pinch up). Hold the pen with one hand, in palm. Your thumb should not be on the button of the delivery device. Insert the pen needle STRAIGHT into the skin.

5. Inject your dose following the instruction for use of the pen injector. After injection, leave the needle in the skin for 10 seconds to ensure the dose accuracy.

6. Put back the outer cap to the needle.

7. Unscrew the needle with outer cap.

8. Discard the used pen needle into a sharps container in accordance with the local guidelines.

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EN

Intended population: Adults and pediatrics over 2 years old.

Intended user: The device can be used by health care professionals and lay users.

Indications: Pen needles are indicated to be used with compatible pen injector device per ISO 11608-1. The list of widely used pen injectors compatible with Pen Needles is included on the packaging.

Storage conditions: Storage conditions for the Pen Needles: -20°C to 65°C, 10%RH to 93%RH (Relative Humidity)

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Warnings

- Consult your doctor if you have any problems for use of this product.
- Select the appropriate size of pen needle according to the doctor's suggestion.
- Please carefully read the instruction for use of the pen injector used with the pen needle.
- Do not use the pen needle if the sealing paper is damaged.
- The period of validity of pen needle is 5 years. Do not use the product beyond the use-by date.
- The pen needle is disposable and use a new needle before each injection.
- Before each injection, ensure the liquid could pass the needle tubing smoothly according to the Step 3 of use method.
- Cover the used pen needle with outer cap and discard the used pen needle into a sharps container according to local regulations. Avoid the risk of cross infection caused by needle puncturing.
- Keep it away from the children and out of their reach.
- Do not reuse the pen needle. Reuse may cause the needle to break, increase injection pain, and cannot guarantee the sterility of the needle.
- If you are punctured by a pen needle already used by others, please consult a doctor promptly.
- Do not re-sterilize the pen needle.

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* Separate waste collection. Check your local municipal guidelines

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Ascensia Diabetes Care Singapore
Pte Ltd
600 North Bridge Road
#06-05 Parkway Square
Singapore 188778
Email: contourcustomerservice@ascensia.com

Distributed in Singapore by:
Pharmaforte Singapore Pte Ltd
6 Tagore Drive
#03-11 Tagore Building
Singapore 787623
Customer service no.: (65) 6452 8488

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Sandstone Medical (Suzhou) Inc.
Building 6, No.168 PuTuoShan Road,
New District, 215153 Suzhou, Jiangsu Province,
PEOPLE'S REPUBLIC OF CHINA
www.sandstonemed.com

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Limited
EU REP Emergo Europe B.V.